



iD For Men Fitting Guide

iD for men are ideal for managing light urinary incontinence in men. The male-specific design enhances comfort and security, while the shape and adhesive strip ensure a secure fit. These products should always be worn with the user's own close fitting underwear.



Step 1

Open the pad and fold in half lengthways then remove the paper from the adhesive strip.



Step 2

Ensuring the wider part of the product is at the front, place the pad inside the underwear, making sure the adhesive strip is securely fastened to the underwear.



Step 3

Finally, pull up the underwear and adjust for comfort and security.

Important Advice

Oil based barrier creams may reduce the performance of the pads. If barrier creams must be used, they should be applied sparingly and only on treatable areas. Store in a cool, dry place. Used products should be disposed of as per local guidelines. Never Flush down the toilet.



iD



iD

**Have you visited
www.id-direct.com yet?**

- **Informative blog posts to support continence care.**
- **Online fitting and skincare guides.**
- **Activate your home delivery order.**
- **Purchase additional products and complementary items.**

**Use code 'Discount10' for
10% off your first order!**

Ontex Healthcare UK
Weldon House, Corbygate Business Park,
Priors Haw Road, Corby, NN17 5JG
01536 269744

Ontex