



## iD Form Fitting Guide

iD Form are anatomically shaped pads providing high levels of absorption, and are available in three lengths.




### Step 1

Start with the net pants pulled up to above the knee, with the top folded in preparation for securing the pad. **Gently open, then fold, the pad in half lengthways** to form a cupped shape, this activates the anti-leak cuffs.




### Step 2

Keeping the pad folded, pass the pad between the legs from front to back with the backsheet turned away from the skin. **Centre the pad, using the wetness indicators as a guide.** Ensure the larger part of the pad covers the buttock area and the smaller area covers the front. If the client is lying down, gently roll them onto their side and position in same way.

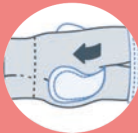

### Step 3

Ensure the front of the pad is fitted up towards the lower abdomen and the back of the pad towards the cleft of the buttocks. **The anti-leak cuffs should fit closely at the groin.** If positioned correctly the front of the pad will sit slightly lower at the front than the back.

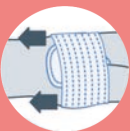

## Step 4



Pull up the pants to fit securely over the pad, ensuring the pants are fitted well at the top of the legs and that the pad remains closely fitted at the groin. **Smooth out the product, ensuring it is crease free.** Adjust the pad and pants as required to ensure the user feels comfortable and secure.

## Wetness Indicator

Each pad has two wetness indicator lines to show the presence of urine, and are a guide to when the product needs changing. They can be checked without the need to remove the pad. The yellow line will turn blue and the blue line/writing will fade. When approximately two thirds of the wetness indicator lines have changed colour the pad is ready to be changed. If there has been any faecal soiling then the pad should be changed immediately.

## Important Advice

Oil based barrier creams may reduce the performance of the pads. If barrier creams must be used, they should be applied sparingly and only on treatable areas. Store in a cool, dry place. Used products should be disposed of as per local guidelines. Never Flush down the toilet.





**iD**

**Have you visited  
[www.id-direct.com](http://www.id-direct.com) yet?**

- **Informative blog posts to support continence care.**
- **Online fitting and skincare guides.**
- **Activate your home delivery order.**
- **Purchase additional products and complementary items.**

**Use code 'Discount10' for  
10% off your first order!**

Ontex Healthcare UK  
Weldon House, Corbygate Business Park,  
Priors Haw Road, Corby, NN17 5JG  
01536 269744

**Ontex**