



## iD Slip Fitting Guide

iD Slip offers an effective solution for managing incontinence, with a high level of absorption, a range of sizes and are available in Cotton Feel (TBS) and with a Plastic Back Sheet (PE).

**Ensure the correct size by measuring the user's waist in line with the umbilicus (belly button). Never assume the measurement. Remember, bigger is not better!**



### Step 1

Unfold and open out the product. Gently open then fold the pad in half lengthways to form a cupped shape, this activates the anti-leak cuffs.



### Step 2

Keeping the pad folded, pass the pad between the legs from front to back. Open the pad out over the buttocks and lower abdomen, ensuring the side with tapes is placed at the back. Align with the waist and centre correctly using the wetness indicators as a guide.



### Step 3

Ensure the anti-leak cuffs are fitted closely into the groin. Fasten the fixation tapes. It is important to create a seal around the legs and groin to reduce risk of leakage.

### **Step 3 - Lying Down**

Fasten the resealable tapes on the upper most side, and then gently roll them and secure the tapes on the opposite side. Now position the user on their back and ensure the anti-leak cuffs are fitted close at the groin. It is important to create a seal around the legs and groin to reduce risk of leakage.

### **Plastic Back Sheet (PE) – Coloured Resealable Tapes**

There are two main parts to the fixing tapes used on All-In-One products. The blue section is the primary tape and once fixed will remain in place. Attempting to remove this tape once fixed will tear the pad.

The secondary section is the white upper tape, which should always be placed on top of the blue tape. You may adjust and removed the white tape, by holding the blue tape with one hand, whilst lifting and removing the white tape with the other hand. You may re-fix the white tape back onto the blue tape as many times as you require.

### **Cotton Feel Back Sheet (TBS)**

iD Slip TBS has four resealable tapes, which can be adjusted as and when needed.



## Wetness Indicator

Each pad has two wetness indicator lines to show the presence of urine, and are a guide to when the product needs changing. They can be checked without the need to remove the pad. The yellow line will turn blue and the blue line/writing will fade. When approximately two thirds of the wetness indicator lines have changed colour the pad is ready to be changed. If there has been any faecal soiling then the pad should be changed immediately.

## Important Advice

Oil based barrier creams may reduce the performance of the pads. If barrier creams must be used, they should be applied sparingly and only on treatable areas. Store in a cool, dry place. Used products should be disposed of as per local guidelines. Never Flush down the toilet.

For online fitting and skincare guides, informative blog posts and to purchase additional products, please visit [www.id-direct.com](http://www.id-direct.com).