



iD Pants Fitting Guide

iD Pants are designed to allow users to maintain their independence. These products combine comfort and discretion due to their elasticated waist and Cotton Feel backsheet.



Step 1

Ensure the correct size by measuring the waist level with the umbilicus (belly button). Start by folding the pants in half lengthways to activate the anti-leak cuffs.



Step 2

Ensuring the sticker is at the back, step into the pants and pull them up, adjusting for comfort and security.



Step 3

Remove the pants by pulling them down. If this is not possible, the pants can be removed by tearing along the side seams.



Step 4

Once removed, simply roll-up the pad and dispose of as per local guidelines.

Important Advice

Oil based barrier creams may reduce the performance of the pads. If barrier creams must be used, they should be applied sparingly and only on treatable areas. Store in a cool, dry place. Used products should be disposed of as per local guidelines. Never Flush down the toilet.





iD

**Have you visited
www.id-direct.com yet?**

- **Informative blog posts to support continence care.**
- **Online fitting and skincare guides.**
- **Activate your home delivery order.**
- **Purchase additional products and complementary items.**

**Use code 'Discount10' for
10% off your first order!**

Ontex Healthcare UK
Weldon House, Corbygate Business Park,
Priors Haw Road, Corby, NN17 5JG
01536 269744

Ontex